


PURESITE - Dust Cover Cap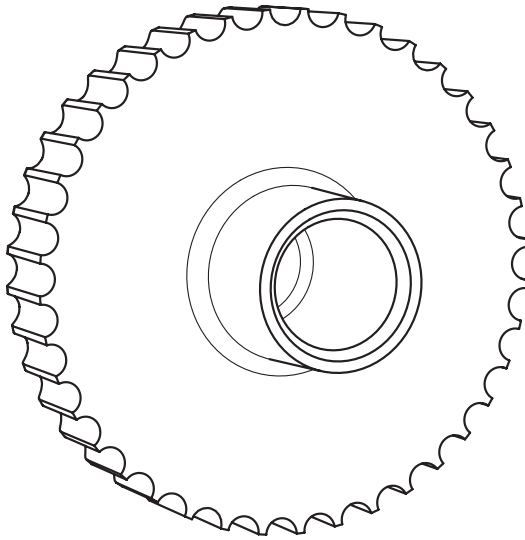

B | BRAUN

GB	Instructions for use	6
DE	Gebrauchsanweisung	7
BG	Инструкция за употреба	8
CN	使用说明	9
CZ	Návod k použití	10
DK	Brugsanvisning	11
EE	Kasutusjuhend	12
ES	Instrucciones de uso	13
FI	Käyttöohje	14
FR	Mode d'emploi	15
GR	Οδηγίες χρήσης	16
HR	Upute za korištenje	17
HU	Használati útastás	18
IT	Istruzioni per l'uso	19
LT	Naudojimo instrukcija	20
LV	Lietošanas instrukcija	21
NL	Gebruiksaanwijzing	22
NO	Brugsanvisning	23
PL	Instrukcja użytkowania	24
PT	Instruções de utilização	25
RO	Instrucţiuni de utilizare	26
RU	Инструкция по применению	27
SE	Brugsanvisning	28
SI	Navodila za uporabo	29
SK	Návod na použitie	30
TR	Kullanım Kılavuzu	31

Handling PureSite - Dust Cover Cap

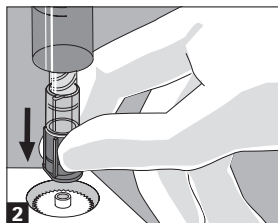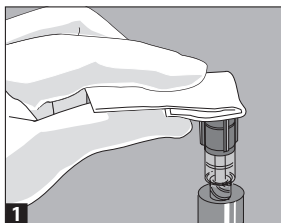**LOT**

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GB Instructions for use

Please read carefully!

Materials used: TPV (Thermoplastic vulcanizate)

CAUTION: Ensure medications are prepared and administered in compliance with drug manufacturer's recommendations.

NOTES:

- Use aseptic technique.
- Single Use Only, Do not resterilize. Use only if the packaging is intact. The Dust Cover Cap is intended to protect PureSite device from touch contact contamination.

WARNINGS:

- Recap PureSite with a new sterile dust cover cap after each use.
- The dust cover cap cannot be used to prime or open the fluid path of PureSite Closed Male Connector.
- Dust cover cap is designed for use with PureSite Closed Male Connector
- Follow Drug Manufacturer's Package Insert for the preparation, administration, storage, and disposal of any drugs utilized with PureSite and dust cover cap.
- Disposal of components should be in accordance with local regulations and procedures for the handling of hazardous drugs.

INSTRUCTIONS FOR USE:

1. Swab PureSite before application as per institutional protocol **(1)**.
2. Open Dust Cover Cap package without touching the cap.
3. To cap PureSite Closed Male Connector, push the center of the male luer (orange grip) into the center tube of the dust cover cap **(2)**.
4. Remove and dispose the dust cover cap before fluid administration. Do not reuse the dust cover cap.
5. Dispose of PureSite, Dust Cover Cap, or associate materials used with any hazardous medications in accordance with the Drug Manufacturer's Package Insert and/or institutional protocol.

Warning

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

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B. Braun Melsungen AG